



SYSTEM OF NARCOTICS IDENTIFICATION

INSTRUCTIONS FOR USE

TEST F - ACID NEUTRALIZER

FOR: Neutralizing Acids contained in All NIK® Test Packs

Before discarding used test packs, remove clip and add one measure of acid neutralizer from Pack F. Add slowly to prevent bubbling over. Do not reseal test pack until effervescence has stopped completely.

Reseal the test pack with the clip and discard in a tamper-free disposal unit.

**ACID
NEUTRALIZER
SODA-ASH
MIXTURE**

1

2

3

4

5

_____ **ONE MEASURE**



13386 International Parkway
Jacksonville, Florida 32218
904.485.1836 FAX: 904.741.5407
© 2105 Safariland, LLC - 1004915

A Brand of the Safariland Group