



# SYSTEM OF NARCOTICS IDENTIFICATION

## INSTRUCTIONS FOR USE

### TEST V - FROHDES REAGENT

FOR: Talwin® (Pentazocine)

Talwin is a registered trademark of  
Winthrop Laboratories

#### Test Function

1. This reagent system presumptively identifies Talwin.

#### How to Use Test V

1. Remove clip and insert into the test pack an amount of powdered suspect material that would fit inside this circle.  Reseal with clip and tap gently to assure material falls to bottom of pack.
2. With the printed side of the test pack facing you, break the ampule. Break by squeezing the center of the ampule with the tips of the thumb and forefinger.
3. Break the left ampule. Agitate gently for one minute. A blue color will develop within 2-3 minutes if Talwin is present.
4. Break right ampule and agitate gently. An immediate yellow color will develop if Talwin is present.

#### Interpretation of Resulting Colors

1. A developing blue color presumptively identifies Talwin.

#### CAUTION

- A. Since these tests contain strong acids, it is suggested that Test F - Acid Neutralizer be used after testing and before disposal.
- B. Before discarding used test packs, remove clip and add one measure of acid neutralizer from Test F.
- C. Add slowly to prevent bubbling over.
- D. DO NOT reseal the test pouch until effervescence has completely stopped.
- E. Reseal pouch with clip and discard in a tamper-free disposal unit.
- F. No attempt should be made to crush the glass particles after the ampoules are broken.
- G. Antidote: In case of contact, immediately flush eyes or skin with water for at least 15 minutes while removing contaminated clothing and shoes. If swallowed, do not give emetics. Contact a physician.

#### WARNING

These tests are NOT designed for use with liquid samples.



13386 International Parkway  
Jacksonville, Florida 32218  
904.347.1200 FAX: 904.741.5407  
© 2019 Safariland, LLC

A Brand of the Safariland Group